


Tetric EvoCeram®

1

Self-Etch

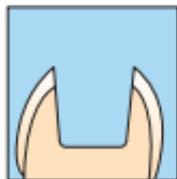

or


Total Etch



15-30 s


10-15 s

Etch & Rinse

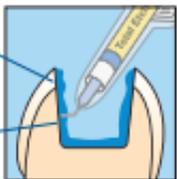

or


Total Etch

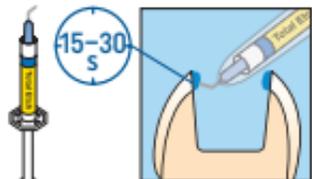

15-30 s

2

Adhese® Universal
VivaPen®

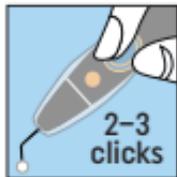

2-3 clicks



20 s



AIR




Bluephase®
PowerCure



3

Optional
Tetric EvoFlow®

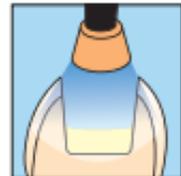


or



Tetric®
PowerFlow




up to
4 mm

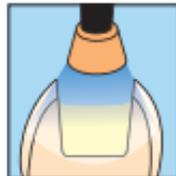

4

Tetric
EvoCeram®

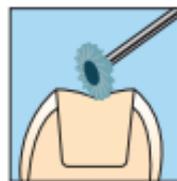

mW/cm ²	Tetric EvoCeram/ Tetric EvoFlow	Tetric PowerFlow	Adhese Universal
500 – 900	20 s	20 s	10 s
1,000 – 1,300	10 s	10 s	10 s
1,800 – 2,200	5 s	5 s	5 s
2,700 – 3,300	–	3 s	3 s

For permanent teeth only

For deciduous teeth see instructions for use

CE 0123

see
Instructions
www.ivoclar.com/elFU

Rx ONLY

For dental use only!

Date information
prepared: 2023-07-10
758273/Rev. 1



Manufacturer: Ivoclar Vivadent AG
Bendererstrasse 2, 9494 Schaan/Liechtenstein

www.ivoclar.com

ivoclar